[EN] Consumer information

Stratacel is an advanced flexible film forming wound dressing for the repair of damaged or compromised skin following fractional skin injuries.

Stratacel is a gel, making it easy to apply to large exposed surface areas and contoured skin, like the face, neck and chest.

When used as directed Stratacel dries to form a protective layer that is gas permeable and waterproof which hydrates and protects compromised skin areas and superficial wounds from chemical and microbial invasion.

Intended use

Stratacel can be applied to open wounds and areas where skin integrity has been compromised.

Stratacel is used to relieve itching and discomfort, and to reduce redness and discoloration associated with damaged skin.

Stratacel was developed to be used as a wound dressing for the repair of damaged or compromised skin.

Stratacel can be used in post cosmetic procedures such as:

- Laser procedures (fractional)
- Chemical peel (superficial)
- Tattoo removal (fractional)
- Microblading
- Microneedling
- Radiofrequency (fractional)
- Vaginal rejuvenation
- Other fractional devices

Stratacel can be used post medical procedures such as:

- Cryosurgery (actinic keratosis)
- Post 5-FU therapy
- Medical tattooing (i.e. post breast reconstruction)
- inflammatory skin alterations
- 1st and 2nd degree burns (with epidermis)

Stratacel can be used with or without a secondary protective dressing.

Stratacel may also be used in conjunction with other adjunctive treatments to improve overall results.

Stratacel contains no alcohols, parabens or fragrances, and is transparent and odorless making it suitable for everyone including people with sensitive skin.

Stratacel may be used during pregnancy and while breastfeeding.

How Stratacel works

When you have undergone any of the procedures mentioned above, the outer layer of your skin (epidermis) has been damaged. This outermost layer (stratum corneum) acts as the protective barrier of the epidermis. When this barrier is broken, water can evaporate more easily. It can no longer protect the skin from the environment which may cause dryness, itching, redness, discomfort, infection and increases the risk of abnormal scarring.

Stratacel dries and recreates a barrier to stop the evaporation of water and the risk of external contamination. This newly created protective layer is gas permeable and waterproof, but does not adhere strongly to the wound bed or the newly formed granulating tissue.


Stratacel helps to promote a moist wound healing environment for compromised skin areas, superficial wounds and burns.

This environment leads to faster re-epithelialization and a reduced inflammatory response, which decreases the recovery time post-procedure.

Directions for use

On first use of 10g and 20g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50g tubes remove the cap, then the protective seal and close with the cap after use.

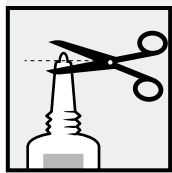Ensure that the affected skin or superficial wound is clean. Gently pat dry as much excess exudate or wound fluid from the area as possible prior to the gel application. Apply a very thin layer of Stratacel directly to the affected area and allow the gel to dry. Stratacel should dry in 5-6 minutes.

If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue. If a secondary dressing is used, drying time is not necessary. Once dry, Stratacel may be covered by sunscreen, cosmetics or clothing.

Stratacel should be applied once daily, or twice daily to exposed areas or as advised by your physician.

For best results Stratacel should be in continuous contact with the skin (24 hours a day / 7 days a week).

Stratacel is intended for single patient use.

Recommended duration of treatment

Stratacel is recommended as immediate post-procedure treatment and should be continued until the skin has fully recovered.

For resurfacing and ablative procedures, continued use is recommended for a minimum treatment of 30 days (24 hours a day / 7 days a week) or until no further improvement is seen.

How much Stratacel do I need?

Stratacel gel is a unique formulation which requires substantially less product per application than typical moisturizing creams or barrier ointments.

- **Stratacel 10g (0.35 oz)** is enough to treat an area of 6 x 6 cm (2.5 x 2.5 inch) twice per day for over 60 days.
- **Stratacel 20g (0.7 oz)** is enough to treat an area of 6 x 12 cm (2.5 x 5 inch) twice per day for over 60 days.
- **Stratacel 50g (1.75 oz)** is enough to treat an area of 12 x 15 cm (5 x 6 inch), twice per day for over 60 days.

Caution

- For external use only.
- Stratacel should not be placed in contact with the eyes.
- Stratacel should not be applied over topical medications unless advised by your physician.
- Stratacel may stain clothing if not completely dry.
- If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- Should your wound show signs of infection or fail to heal after 30 days of initial product application consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for highly exudative wounds, tunneling wounds or 3rd degree burns.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Do not use if the tube is damaged.

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Stratapharma AG